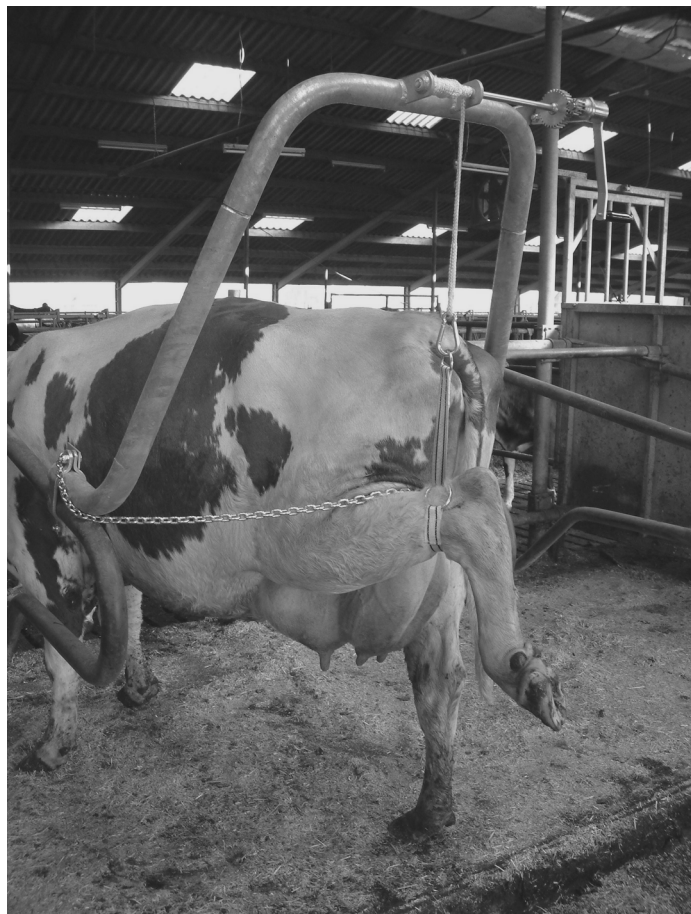
Vink Hoefbekapbeugel

Vink Hoofcareframe

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Vink Klauenpflegestand

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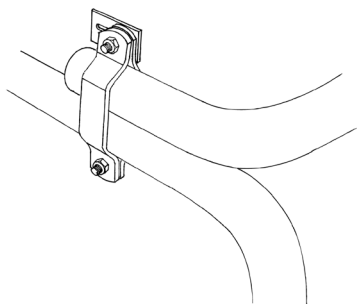
Introduction

The Vink Hoofcare frame allows you to construct an easy to use hoofcare box, which enables immediate treatment of hoof ailments of the hind legs. If an easy to use hoofcare box is not available, treatments are often postponed. As a result animals frequently have to be culled due to lameness, which could have been cured if they had been treated in time. The Vink Hoofcare frame can be attached to any kind of partition in the open stall. The cow is treated in an easy to reach spot and in an environment that is familiar to her. The hoofcare frame utilizes a completely galvanized self-braking safety winch and is supplied with rope, heel strap and all necessary attachment materials.


Assembling the hoofcare frame in the stable

- Find a spot that is easy to reach, for example in the vicinity of the milking stable.
- Screw the provided parallel clamps on the back of the partition. Both at the left and at the right side in such a way that the clamps are loose and the tubes of the hoofcare frame can still slide into them. (see drawing)

The attachment of the cow safety chain should be facing the front, the plate with the open slit to the right and the plate with the closed slit to the left. Both plates at the upper/inside of the frame.

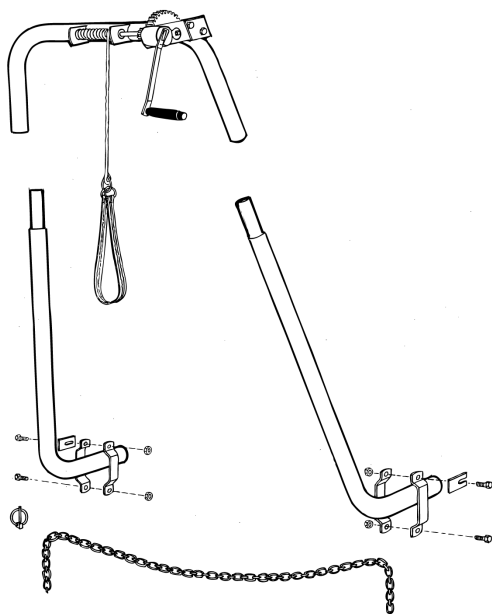- Assemble the hoofcare frame as shown in drawing

- Place the hoofcare frame on the partition with the crank at the right side and slide the clamps over the tube ends.
- Slide the hoofcare frame backwards in such a way that the bend of the partition is aligned with the bend of the hoofcare frame (see photo 3)
- Tighten the 4 M12 bolts (spanner size 19)
- Attach the cow safety chain to the left side of the partition using the safety clip.

The hoofcare frame is now ready for use.

Special attachment sets are available for floating partitions, which prevent the box from sagging. The diameter of the tubes of the partition determines which parallel clamps need to be used; 1.5" or 2". The frame fits partitions varying in width from 105 cm (41.3 inch) to 125 cm (49.2 inch)

Safety rules

Functioning of the safety winch:

Lifting the leg – turn crank clockwise

Lowering the leg – turn crank anticlockwise

Always use the cow safety chain to prevent the cow from escaping. Always be prepared for unexpected movements of the cow.

Using the hoofcare frame

- Put the cow that needs to be examined in the box
- Use the cow safety chain to secure the cow
- Attach the strap just above the heel of the animal
- Attach the heel strap to the hoisting rope with the snap shackle
- You are now able to lift the hind leg by turning the crank clockwise

Repair

Replacing the hoisting rope

- Unwind the entire rope from the winch and detach the clamp that attaches the hoisting rope. Assemble the new hoisting rope and wind it up the winch clockwise.

Replacing the ball bearings or winch

- Remove the hoisting rope; disassemble the winch by removing the support (nr.26). Knock the nylon bearings out of their supports and knock new ones in. Slide the winch through the bearings and screw the support back on.

Replacing the crank

- Remove the 2 M 6 bolts (spanner size 10) from the front of the winch. In order to assemble a new crank, the entire drum that is connected to the crank to will have to be turned left. Place the new crank and the holding plate and assemble the 2 M 6 bolts. Tighten the bolts (not too tightly) and unscrew them a quarter of a turn, enabling them to move freely in the grooves. Bend the holding plate over the heads of the bolts to prevent them from coming loose.

Parts

5	Crank	HBB05
6	Handle	HBB06
7	chain	HBB07
8	Locking pin	HBB08
11	Axe bearing	HBB11
15	Frame	HBB15
22	Winch	HBBL03
24	Hoist-Rope	HBBK
25	Hoist-Strap	HBBB
26	Fastening Winch	HBB26
27	Fastening set 1.5"	HBBM1
29	Fastening set 2"	HBBM2
30	chain plate (set)	HBB30

If any questions arise about the functioning of the apparatus, please contact your dealer or the manufacturer.

Vink Warranty

Vink warrants, the Vink hoof care spray will, with normal use and service, be free from faulty parts, manufacture or workmanship for a period of one (1) year from invoice date.

The warranty does not apply where damage is caused by other factors, including without limitation

- Normal wear
- Abuse, mishandling, accident or failure to follow operating instructions
- Repair by unauthorized person
- Damage caused by external influences as floods, fire etc. environmental and atmospheric disturbances and mechanical damage caused by dropping

This warranty is limited to the exchange of the defective part during its warranty period, Disassembly and assembly of parts are for the Client's account

The part/product must be returned to Vink carriage paid.

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Hersteller: Vink-Elst B.V.
Manufacturer: Vink-Elst B.V.
Fabrikant: Vink Elst B.V.,