Original  
Manual

---

9028291

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GB: Calf Puller

DK

S

D

GB

[www.p-lindberg.dk](http://www.p-lindberg.dk)

[www.p-lindberg.no](http://www.p-lindberg.no)

[www.p-lindberg.se](http://www.p-lindberg.se)

[www.p-lindberg.de](http://www.p-lindberg.de)

**Kontakt Danmark:**

P. Lindberg A/S  
Sdr. Ringvej 1  
6600 Vejen  
Tlf. 70 21 26 26  
Fax 70 21 26 30  
salg@p-lindberg.dk  
www.p-lindberg.dk

**Kontakt Norge:**

P. Lindberg  
Doneheia 62  
4516 Mandal  
Telefon: 21 98 47 47  
hei@lindberg.no  
www.p-lindberg.no

**Kontakt Sverige:**

P. Lindberg Sverigefilial  
Myrangan 4  
745 37 Enköping  
Tel. 010-209 70 50  
order@p-lindberg.se  
www.p-lindberg.se

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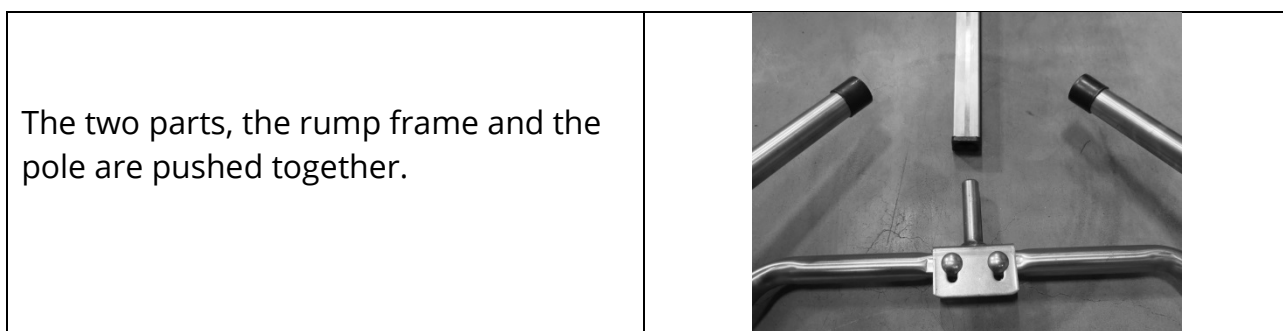
**Calfpuller – Item no. 9028291**

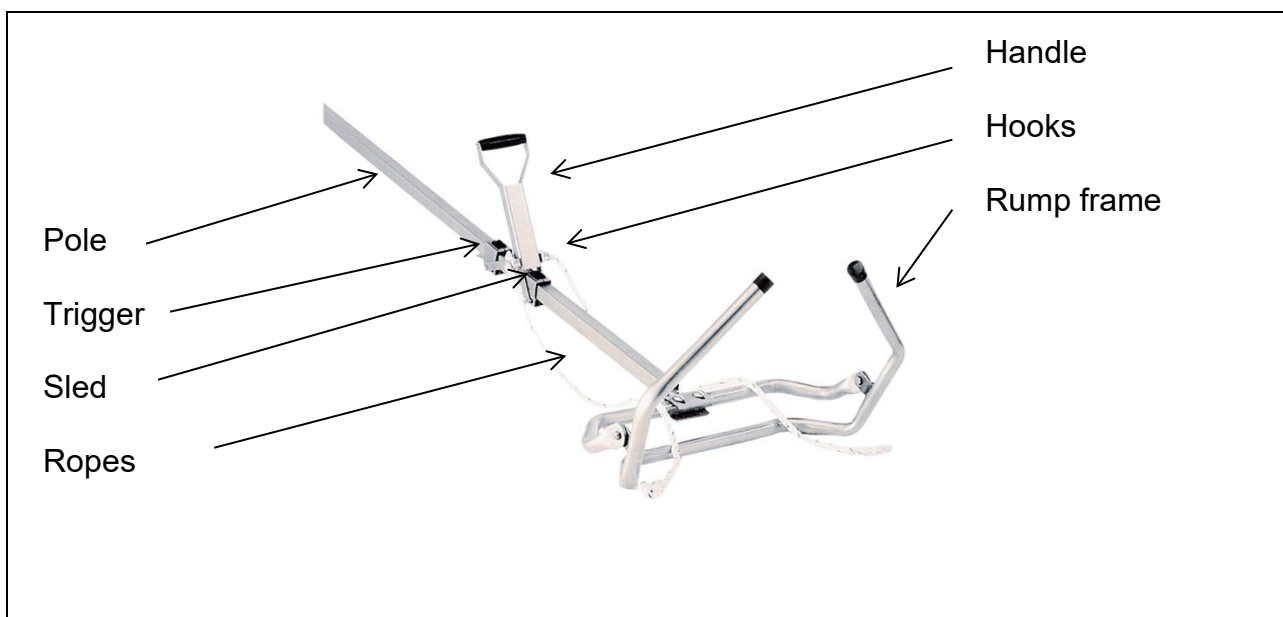
**Description:** Calfpuller constructed in stainless steel and with safety release. Trigger at max. pull of 155 kg.

**Intended use:** May only be used as described in the instructions for use, all other use is considered incorrect.

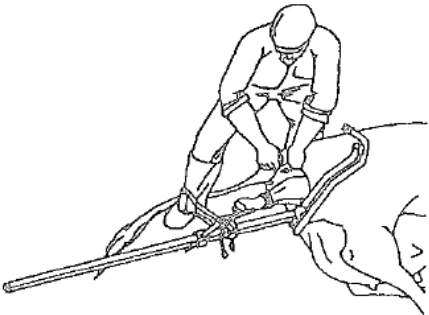
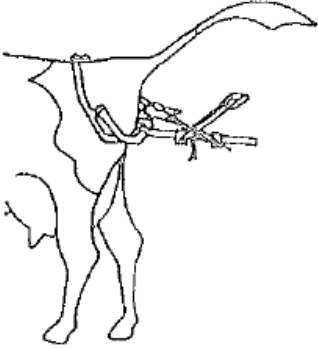
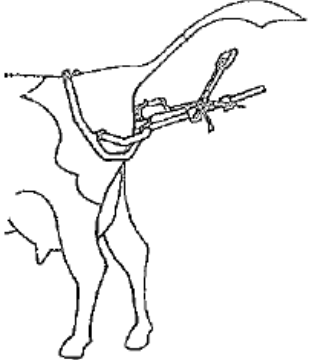
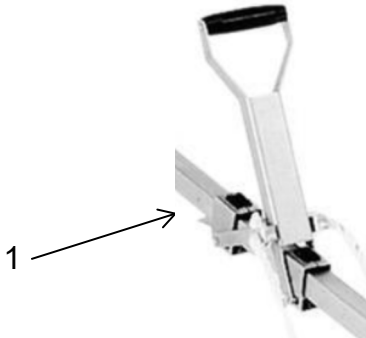
**Assembly**

**Overview**

## Instructions

<p>The calf puller is placed behind the cow. The opening of the rump frame should point upwards.</p> <p>The ropes are put first on the calf's legs and then on the pull hooks on the sled.</p> <p>Make sure that the pull ropes are equally tight for the same pull in both legs.</p> <p>Gently pull on the pull handle. The calf puller adjusts in place no matter if the cow is standing or lying down.</p>	
<p><b>Normal birth.</b></p> <p>Pull obliquely downwards until the calf's head is born. Then pull horizontally.</p> <p>When the calf's pelvis is to be born, pull it down again at an angle.</p>	
<p><b>Posterior birth.</b></p> <p>Pull obliquely upwards until the hind legs and tail are born. Then pull horizontally.</p>	
<p>To loosen the ropes, activate the trigger (1) and push the handle towards the cow.</p> <p>If the ropes are very tight, pull the handle back a little before activating the trigger and move the handle back towards the cow.</p>	

## Warning

Make sure there is enough space around the cow. A standing cow in birth can suddenly lie down. The rod can strike violently to one of the sides.

We reserve the right to change the technical parameters and specifications of this product without prior notice.